
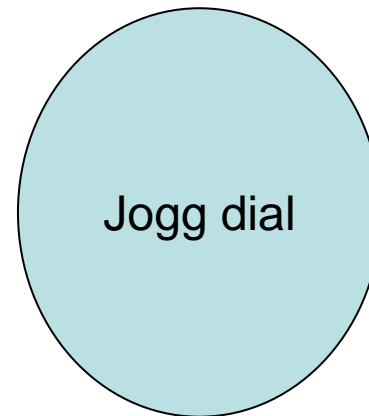


Service mode

stisni paralelno

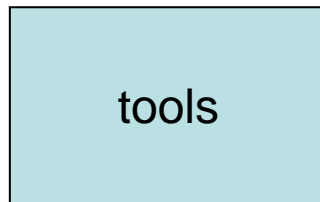

utipkaj 3641 i pritisni CONFIRM

Energy Adjust

stisni i drzi

- Test Energy Adjust


stisni

NO


stisni

MRK je ploca sa lijeve strane

Adjust Offset

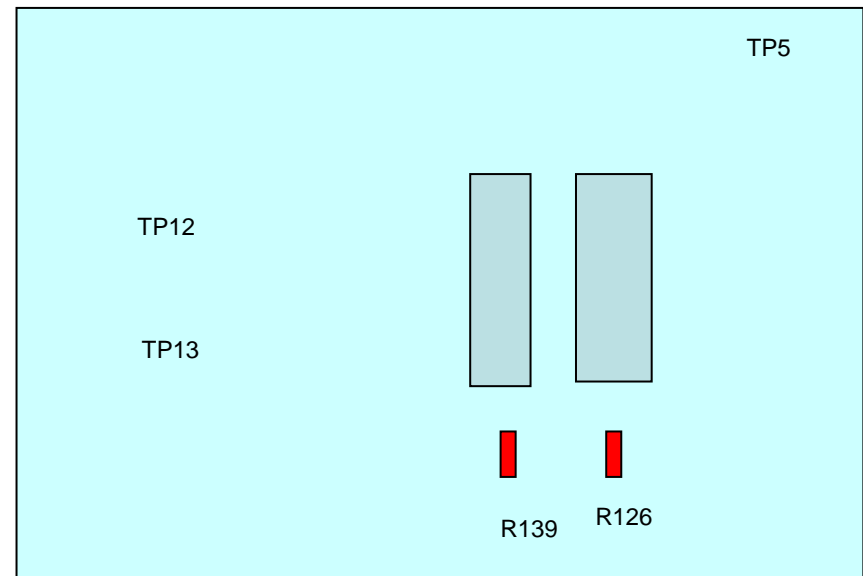

Adjust Gain

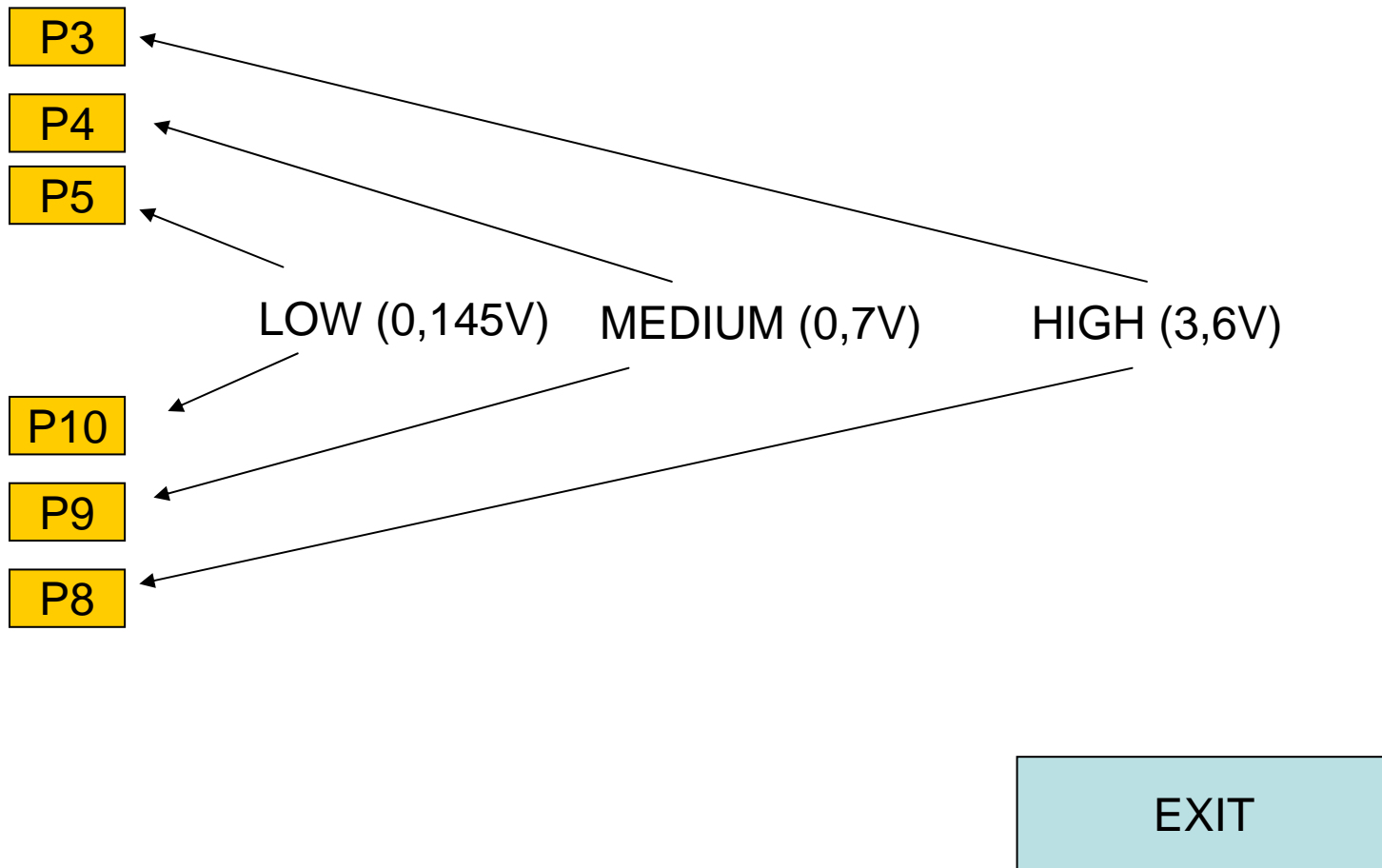

Energy Adjust Press

YES

press

Chanel A

Drive

Chanel B

Adjust 0,4J

adjust drive (aprox. 900mV) on display till on external energy meter reading is 0,4J

Confirm & Next

press

energy adjust page 5

- 2J (around 1,8V) CONFIRM
- 2J GAIN MEDIUM (around 1V) CONFIRM
- 10J Gain MED (around 2,1V) CONFIRM
- 10J Gain LOW (around 1V) CONFIRM
- 50J Gain LOW (around 2,1V) CONFIRM

- EXIT