

MedioStar XT / miXT : Skin Rejuvenation upgrade



1. Skin Rejuvenation 14mm Tipp (0,5-1 Hz)

skin type I - III: 145-250 ms, 25-32J/cm²,

skin type IV: 300-350 ms, 25-30 J/cm²

single impulse

- 2-4 passes with additional 2 passes over deep wrinkles
- 5 session in total (every 3 days)
- treatment can be repeated after 3 months
-

2. Skin Tightening

12mm/14mm Tipp (0,5-1Hz)

skin type I - III: 250-400 ms, 30-40J/cm²,

skin type IV: 300-350 ms, 30-35 J/cm²

single impulse

- 1-2 passes with additional pass over wrinkles
- 5 session in total (every 5 days)
- Treatment can be removed after 3 months

3. Acne

12mm/14mm Tipp (1Hz)

skin type I-IV: 150 ms, 25(male)-35(female) J/cm²

single impulse

- 1-2 passes
- repeat after 3-4 days
- repeat once every month related to intensity of acne problem

4. Skin whitening / pigment

4mm/6mm Tipp for pigment

14mm for whitening

skin type I - III: 100-300 ms, 30-120 J/cm²

skin type IV: 200-250 ms, 25-40 J/cm²

double impulse/ single pulse