

GentleLASE

(MGL)

Service Manual

Models: 9914-00-0880

9914-00-0890

9914-00-0910

9914-00-0930

*Candela Corporation
530 Boston Post Road
Wayland, MA 01778-1886
Telephone (508) 358-7637*

8501-00-1741, Revision A
July 2003

Toll Free (800) 733-8550 ext.336 (Technical Assistance)
Toll Free (800) 73-LASER (Customer Service)

Caution

Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure.

Caution

Federal (USA) law restricts this device to sale by or on the order of a physician.

Copyright 2002. All rights reserved. Printed in USA.

"Candela"[™]

TABLE OF CONTENTS

SECTION ONE	THEORY OF OPERATION	Revision
8501-00-1762	Theory of Operation, GentleLASE	A
SECTION TWO	FIRMWARE DOCUMENT	
1010-01-3020	Firmware Design Document, GentleLASE	B2
1010-06-3020	Firmware Requirement Specification	B2
SECTION THREE	SERVICE PROCEDURES	
8503-01-0060	Inspection and cleaning of Optics	02
8503-01-0821	Installation Procedure.....	A
8503-01-0822	Routine Preventive Maintenance Procedure	A
8503-01-0560	Flashlamp Replacement Procedure	C
8503-01-0570	Laser Head Replacement	D
8503-01-0600	Handpiece Lens Replacement procedure	C
8503-01-0823	Temperature Calibration Procedure	A
8503-01-0776	DCD Pressure Verification.....	A
8503-01-0824	High Voltage Calibration Procedure	A
8503-01-0825	Head Detector Calibration Procedure	A
8503-01-0826	Laser Rail Alignment Procedure	A
8503-01-0827	Focus Lens Replacement Procedure	A
8503-01-0828	Aiming Beam Alignment Procedure.....	01
8503-01-0829	Calport Calibration Procedure	A
8503-01-0830	Bubble Sensor Calibration Procedure	A
10-400-00144	GentleLASE System Performance & Usage Data Sheet.....	C
10-040-01502	Form, Temp for Spot Size Verify.....	02
8502-15-0208	Mini-GentleLASE PM Check List.....	A
8502-00-0858	Treatment Guidelines	B

SECTION FOUR	ENGINEERING DRAWINGS.....	Revision
8501-02-1741	DRAWING PACKAGE	A
	Schematics	Revision
7111-80-2230	Touch Panel Interface PCB.....	06
7111-80-2510	HP Bulkhead PCB	01
7111-80-2520	DCD Canister PCB.....	03
7111-80-2610	AC Distribution PCB.....	08
7111-80-2620	HV Bleeder PCB.....	06
7111-80-2630	CPU / I/O PCB.....	03
7111-80-2650	Energy Detector PCB	03
7111-80-2660	Delivery HP PCB	03
7111-80-2670	HP Slider PCB	02
7111-80-2680	HP Switch PCB.....	02
9914-80-0880	System Schematic.....	08
	Diagrams	
9914-84-0880	GentleLASE Functional Block Diagram.....	04
9914-91-0880	GentleLASE Family Tree	A
9914-91-0910	GentleLASE-LE Family Tree.....	04
	Assembly Drawings/BOMs	
7121-00-6700	Bom, Laser Head Assembly	A
7121-99-6700	Laser Head Assembly.....	C
7122-00-3287	Bom, Fluid Section Assembly	B
7122-99-3287	Fluid Section Assembly.....	C
7122-00-3289	Bom, Upper Chassis Assembly,	A
7122-99-3289	Upper Chassis Assembly.....	B
7122-00-3292	Bom, AC Input Assembly.....	A
7122-99-3292	AC Input Assembly.....	B
7122-00-3297	Bom, Calibration Port Assembly,	B
7122-99-3297	Calibration Port Assembly	B
7122-00-3298	Bom, Laser Rail Assembly,	05
7122-99-3298	Laser Rail Assembly.....	02
7122-00-3300	Bom, Fiber Receptacle Assembly,	A
7122-99-3300	Fiber Receptacle Assembly.....	A
7122-00-3301	Bom, Head Detector Assembly,	04
7122-99-3301	Head Detector Assembly.....	02
7122-00-3302	Bom, HV Section Assembly	A
7122-99-3302	HV Section Assembly.....	B

Assembly Drawings/BOMs	Cont.	Revision
7122-00-3306	Bom, DCD Option Assembly,	B
7122-99-3306	DCD Option Assembly.....	A
7122-00-3316	Bom, DI Heat exchanger Assembly	A
7122-99-3316	DI Heat exchanger Assembly.....	A
7122-00-3317	Bom, Heater Manifold Assembly	03
7122-99-3317	Heater Manifold Assembly.....	A
7122-00-3319	Bom, Display Bezel Assembly,	A
7122-99-3319	Display Bezel Assembly.....	A
7122-00-3325	Bom, Focusing Lens Assembly,	02
7122-99-3325	Focusing Lens Assembly	05
7122-00-3326	Bom, Aiming Beam Assembly,	05
7122-99-3326	Aiming Beam Assembly.....	02
7122-00-3327	Bom, Beam Shutter Assembly,	03
7122-99-3327	Beam Shutter Assembly.....	A
7122-00-3333	BOM, Aiming Beam, Diode 650nm	04
7122-99-3333	Aiming Beam, Diode 650nm	A
7122-00-3339	BOM, Distance Gauge Stick, MGL	A
7122-99-3339	Distance Gauge Stick, MGL	02
7122-00-3340	Bom, 6/8/10 mm Delivery System Assembly, w/DCD	C
7122-99-3340	6/8/10 mm Delivery System Assembly, w/DCD.....	03
7122-00-3342	Bom, 12/15/18 mm Delivery System Assembly, w/DCD	C
7122-99-3340	12/15/18 mm Delivery System Assembly, w/DCD	03
7122-00-3309	Bom, 12mm Delivery System Assembly, w/DCD, MGL-LE,	02
7122-99-3340	12/15/18 mm Delivery System Assembly, w/DCD	03
7122-00-3341	Bom, 6/8/10 mm Delivery System Assembly, w/o DCD	B
7122-99-3341	6/8/10 mm Delivery System Assembly, w/o DCD.....	03
7122-00-3343	Bom, 12/15/18 mm Delivery System Assembly, w/o DCD	B
7122-99-3341	12/15/18 mm Delivery System Assembly, w/o DCD.....	03
7122-00-3281	Bom, 12 mm Delivery System Assembly, w/o DCD, MGL-LE,	02
7122-99-3341	12/15/18 mm Delivery System Assembly, w/o DCD.....	03
7122-00-3348	BOM, DI pump MGL	A
7122-99-3348	DI pump MGL.....	A
7122-00-3349	BOM, DCD Valve, MGL	A
7122-99-3349	DCD Valve, MGL.....	B
7122-00-3350	BOM, 6/8/10 mm Lens Cap, MGL	A
7122-99-3350	6/8/10 mm Lens Cap, MGL	A
7122-00-3351	BOM, 12/15/18 mm Lens Cap, MGL	A
7122-99-3351	12/15/18 mm Cap, MGL	A
8901-00-9301	PM Kit MGL	01

SECTION FIVE

REFERENCE

